... atunci cand secundele conteaza

.....
**Reprezentant Medical
Estima Medical Group**

- Prezentare generala Celox
- Studii clinice
- Intrebari frecvente

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- **THE EFFICIENCY OF THE HAEMOSTATIC POWDERS IN THE TREATMENT OF THE HEMORRHAGIC LESIONS**

Mihaela Maxim, P. Grigoraș, G. Sandu, Roberta Brănișteanu, Diana Cimpoeșu, "Gr. T. Popa" University of Medicine and Pharmacy of Iași, Emergency Reception Unit – The Mobile Service of Emergency, Resuscitation and Extrication of "Sf. Spiridon" Clinical Emergency Hospital of Iași

Introduction: The haemostatic powders are granular products made from plants or seafood that coagulate blood, independently from the normal coagulation mechanisms of the body, acting as a sponge that extracts water from the blood and binds directly to the red blood cells.

Objectives: Demonstrating the efficiency of the haemostatic powders applied topically in moderate and severe bleedings, and assessing the clinical characteristics of the patients treated as emergencies.

Method: Retrospective study of the patients with traumatic hemorrhagic lesions admitted in the Emergency Reception Unit – Mobile Service of Emergency, Resuscitation and Extrication of "Sf. Spiridon" Clinical Emergency Hospital of Iași between January 1st and September 15th 2009.

Results

The study lot included 48 patients (male:female rate = 1.82:1)
There were no deceases during the first 24 hours after the event.
The first administration in the north-east area: November 2008.

- **Conclusions:**

The haemostatic powders proved efficient in stopping the moderate and severe posttraumatic bleedings in the hospital, but especially in pre-hospital stage, where the easiness of use and their enhanced efficiency make them preferable to other haemostatic methods.

Age distribution

0 – 16 years

16 – 30 years

> 30 years

Aggressions – 12.5%

Traffic accidents – 41.5%

Domestic accidents – 46%

Distribution of lesions on anatomic segments

Trunk 15%

Head 42%

Limbs 43%

Hemorrhagic shock 29%

Hemorrhage 71%

Place of administration of the haemostatic powders

14.59 %

85.41% pre-hospital administration

Admissions %

31.25%

68.75 % admitted patient

Intrebari?